

**SKID PROOF
EFFECT**



MSTRIPPER

CUTTER & HAND SEALER/ROLLER



- **Stainless Steel Body**
- **Light Weight**
- **Spring Powered Handle**

MSTRIPPER

CUTTER & HAND SEALER/ROLLER

'MITRA' introduces 3-in-1 device - 'Stripper', 'Cutter' and 'Manual Tube Sealer'. It's a handy, Skid Proof tool used for stripping, cutting and sealing of Blood Bag tubes.

...extra features

- Completely stainless steel body, so no chance of rusting ever.
- Light weight
- Spring powered handle. To ensure uniform pressure while pressing to close and automatic recoiling of spring to release handle for opening.
- Screw-less rollers to avoid loosening of the rollers.
- Extra sharp cutting edges.
- Hot melt plastic coated handle covers to ensure no space or gap in between the handle arms and plastic covers.
- Ergonomically designed handle for better gripping.
- Roller guide to avoid any damage of tube.
- Manual tube sealing by aluminum rings.


STEPS TO FOLLOW:

TUBE STRIPPER/ROLLER

- Keep the collection tube in between the rubber roller.
- Press the handle and roll it towards collection bag.

MANUAL HAND TUBE SEALER

- Fold the collection tube.
- Insert the Aluminum ring on it.
- Hold the folded tube with ring in up-down position and press between the grooves at the tip of the hand sealer.

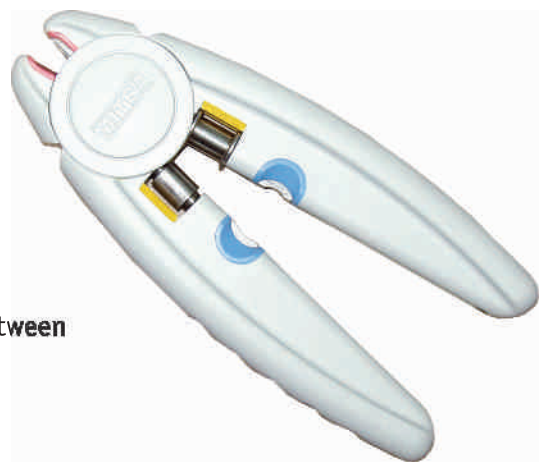

Specification/Designs are subjected to change without any obligation on the part of the manufacturer

For Further Information Please Contact:

Mitra Industries (P) Ltd.

14/4, Delhi-Mathura Road, Near Metro Station, Mewala Maharajpur, Faridabad, Haryana- 121003, INDIA.

Ph.: 0129-2271068, 2271069. **Toll Free No.:** 1800 102 0307

E-mail: customersupport@mipl.in **Website:** www.mitraindustries.com