

EMPTY STERILE EFFLUENT TRANSFER BAG WITH CONNECTING TUBE & COUPLER/CONNECTOR

3 LTR

ACCESSORY FOR CONTINUOUS AMBULATORY PERITONEAL DIALYSIS (CAPD)



Use :

To drain out peritoneal dialysis fluid from peritoneum.

Empty Sterile Effluent Transfer Bag with connecting tube & coupler/connector 3 Ltr.

Precautions:

- Do not Reuse.
- Discard, after use. Do not re-sterilize.
- Do not use if package has been previously opened or damaged.
- Do not use if outer wrapper has any signs of moisture or color change.

For Connecting Empty Sterile Effluent Transfer Bag with connecting tube & coupler/connector 3 Ltr.

Items Required:

- Face Mask.
- Disinfection agent or Liquid Soap for Handwashing.
- Sterile Transfer Set Cap.
- Sterile Drape.
- Sterillium (Hand Scrub).
- Clamps

Procedure:

Please follow CAPD protocols:-

1. Wear the Face Mask.
2. Keep the Doors and Windows Closed.
3. Switch off the Fan and the A/C.
4. Do 6 Steps of hand washing and air dry the hands properly.
5. Clean the Table and clean the outer cover of the 3Ltr effluent bag using sterillium and peel open the outer cover (Make sure the inner bag is not touched).
6. Repeat the 6 steps of hand washing and air dry the hands properly.
7. Use Sterillium and scrub the hands well.
8. Place the transfer set on the Sterile Drape.
9. Carefully remove the 3 Ltr effluent bag from the pouch
10. Hold the Grip of Effluent transfer Bag connector in one hand and using the other hand lift the Transfer set and remove the Cap of the transfer set.
11. Connect the 3 Ltr effluent Bag to the transfer set and slightly twist to tighten.
12. Open the transfer set twist lock and also open the clamp on the drain line for the effluent to drain.
13. Once the drain is completed close the clamp on the drain line.
14. Close the transfer set by turning the twist lock.
15. Use sterillium and clean your hands.
16. Open a new sterile transfer set Cap.
17. Carefully disconnect the drain bag from the Transfer set (Follow instructions as per IFU of Transfer set cap).
18. Close the Transfer set with the new Transfer set Cap.
19. The disconnected 3 Ltr effluent bag with effluent is ready for measurement/Clarity checking/Sample collection.

*Specification/Designs are subject to change without any obligation on the part of the manufacturer.



Manufactured & Marketed by:

Mitra Industries Private Limited

14/4, Delhi-Mathura Road, Faridabad, Haryana-121 003. INDIA

T: +91-129-4099300 | Toll Free: 1800 1020 307

E: customersupport@mipl.in | W: www.mitraindustries.com