

TRANSFER SET CAP (Sterile)

ACCESSORY FOR PERITONEAL DIALYSIS



*Detaching the new cap from the ring



*Unlocking the new cap from the ring



Use :


Transfer set cap is intended for disinfection of the Tip of transfer set and protection from the contamination in external environment by enclosing it.

Precautions :

- For Single Use only.
- Discard, after use.
- Do not re-sterilize.
- Do not use the product if packing found damaged or previously opened.
- Do not use if sponge is dry or color is not golden brown.

Instructions for Use :

1. Wear the mask and do 6 Steps of hand washing as shown in figure1 to 6.

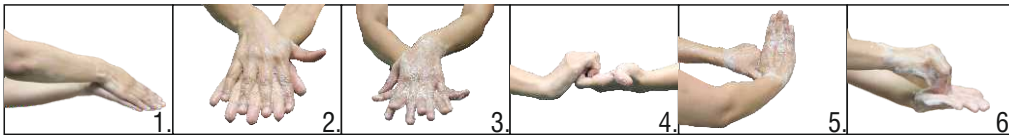

2. Remove the sterile transfer set cap from the Tyvek pouch and hang it in little finger.
3. With the same hand pick the Transfer set by holding the Transfer Set Blue Portion.
4. Using the other hand Lock the Transfer set by turning the Twist lock.
5. Loose the Sterile Transfer Set Cap from the ring cap.
6. By Transfer set Tip pointing downwards disconnect the PD bag from the transfer set.
7. Unscrew the Transfer set cap from the ring and check the Povidine-iodine solution swab inside the cap. It should be golden brown in color.
8. Immediately close the transfer set with the transfer set cap tightly taking care that the transfer set end is not touched (Fig. 7).
9. Change the transfer set cap within 2-3 seconds time.

Cautions: ⚠

- Do not Expose the tip of the transfer set in air too long.
- Change the transfer set cap in 2-3 seconds time once the line is disconnected from the tubing.

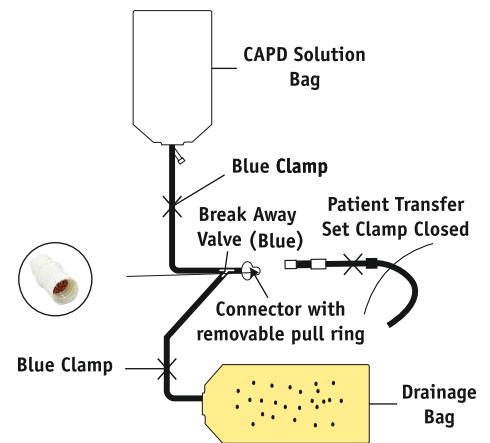


Figure 7