

TRANSFER SET

ACCESSORY FOR PERITONEAL DIALYSIS



Use :

Transfer set is intended to use as tubing connector/extension between the peritoneum catheter/titanium adapter and the peritoneal Dialysis solution bag .

Precautions :

- Transfer set to be changed by PD Nurse/Doctor in PD Centre or Hospital.
- Do not use if outer wrapper has any signs of moisture or color is changed.
- Do not use if package has been previously opened or damaged.
- Discard, after use.
- Do not re-sterilize & re-use.

Instructions for Use:

Procedure:

Wear the face mask and do 6 Steps of hand washing.(Figure 1 to 6)


1. Air Dry the Hands Properly.
2. Clean the Table and Required Materials and arrange them on the table.
3. Do the 6 Steps of hand Wash again and Dry the hands thoroughly. (Figure. 1-6)
4. Open the Transfer set Change Kit and the Transfer Set Pouch.
5. Wear the Sterile Gloves.
6. Clean the Catheter and lock the Line of the Catheter using a clamp.
7. Fill the Bowl with Betadine, Remove the Old line which is attached to the Titanium Adapter, Soak the tip of the Titanium Adapter into the Betadine Solution for 3 Minutes.
8. Change the Gloves and dry the Tip of the Titanium Adapter using a Sterile Gauze.
9. Pick the New Transfer Set and connect to the Titanium Adapter tightly by twisting the tips with both the hands.
- 10.Using one hand, Hold the Transfer Set Blue Portion and lift the Transfer set.
- 11.Using the other hand, Lock the Transfer set by turning the Twist Lock (White Color).
- 12.Open the Transfer set Cap and observe for the wet sponge Povidine-Iodine swab inside the cap and twist close the end of the Transfer set tightly.
- 13.Now the Transfer set is ready for exchanges.

Cautions: ⚠

- This device is to be sold by on the prescription of a registered medical practitioner only.
- Do not Expose the tip of the Titanium Adapter and Transfer set in air too long.
- Change the Transfer Set Cap with in 2-3 seconds time.

Items requires to connect transfer set:

- Transfer Set.
- Transfer Set Cap.
- Transfer Set Change Kit (Forcep/Catheter Clamp/ Bowl/Plastic Sheet/Gauze/Surgical Gloves/Tissue Paper/Povidine-Iodine) – All Sterile.
- Disinfectant agent/Isopropyl Alcohol for hand washing.
- Face Mask for PD Nurse & Patient.
- Sterillium (Hand Scrub).
- Clamps.

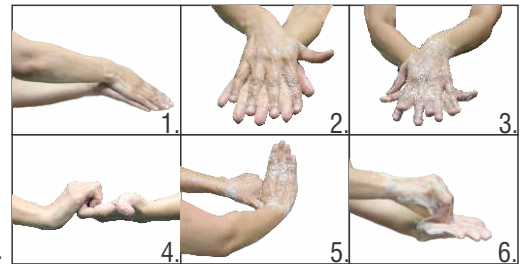

*Specification/Designs are subject to change without any obligation on the part of the manufacturer.