

CLAMP FOR TUBING OF CAPD BAG



ACCESSORY FOR CONTINUOUS AMBULATORY PERITONEAL DIALYSIS (CAPD)

Use:

To support "Drain-Flush-Fill" step during CAPD procedure.

Items required:

- Face Mask
- Disinfectant agent/Isopropyl Alcohol for hand washing
- Tubing clamp



Instructions for Use:

1. To open and close the PD fluid in fill line and in drain line during exchange procedure.
2. Open clamp and place across the tubing as per drawing.
 - a) Press to stop fluid flow in the tube as shown in fig. 1
 - b) Open to allow fluid flow as shown in fig. 2
3. Please follow CAPD exchange protocol.
4. Once the drainage is completed, close the transfer set line with clamp as shown in Fig. 3.
5. Now open the blue tube clamp provided on the tubing of the CAPD solution bag and the connector and allow the new solution to flow into the drainage container while you count from 1001 to 1005 slowly. This is required to flush or remove any air in the primary tube as shown in Fig. 4.
6. Close the drain tubing using the blue tube clamp.
7. Open the transfer set clamp and allow the infusion of CAPD solution into the peritoneum as shown in Fig. 5.
8. On completion of infusion, close the transfer set clamp and then the blue tube clamp on the primary tubing. This will ensure that air does not enter into peritoneum as shown in Fig. 6.

