

## EMPTY STERILE EFFLUENT TRANSFER BAG WITH CONNECTING TUBE & COUPLER/CONNECTOR

### 15 LTR

ACCESSORY FOR AUTOMATED PERITONEAL DIALYSIS



#### Use:

To drain out peritoneal dialysis fluid from peritoneum while using APD (Automated Peritoneal Dialysis) system.

#### Empty Sterile Effluent Transfer Bag with connecting tube & coupler/connector 15 Ltr.

#### Precautions:

- Do not Reuse.
- Discard, after use. Do not re-sterilize.
- Do not use if package has been previously opened or damaged.
- Do not use if outer wrapper has any signs of moisture or color change

#### Connecting Empty Sterile Effluent Transfer Bag with connecting tube & coupler/connector 15 Ltr.

- Face mask.
- Disinfection agent or Liquid Soap for Hand Washing.
- Sterile Drape.
- Sterillium (Hand Scrub).

#### Procedure for Effluent Bag:

Please follow APD protocols:-

- Hold the Grip of Effluent Transfer Bag connector in one hand and Connect the 15Ltr Drain Bag to the transfer set and slightly twist to tighten.
- Open the clamp on the drain line and the 15 Ltr Drain bag is ready to use.

#### Connecting Empty Sterile Effluent Transfer Bag with coupler 15 Ltr. to mCycler

#### Items Required:

- Face Mask.
- Disinfection agent or Liquid Soap for Hand washing.
- mCycler
- Sterile Drape.
- Sterillium (Hand Scrub).
- 5 Ltr Fluid bags for mCycler.

#### Procedure for mCycler:

Please follow APD protocols:-

1. Hold the Tubing in one hand and lock all the clamps on the tubing.
2. Carefully arrange the tubing into the pinch valves on the mCycler as per colour code.
3. Place the 15 ltr effluent transfer bag in the lower Weighing Scale on the mCycler.
4. Use Sterillium to Scrub the hands well.
5. Follow the instructions of the mcycler to connect the tubing.
6. 15 ltr effluent transfer bag is to connect on APD(Automated Peritoneal Dialysis) machine.